

# botiss grafter

**APPLICATOR** FOR BONE SUBSTITUTE MATERIALS



## **THINK SYSTEM:**

ALLOWS APPLICATION OF VARIOUS BONE SUBSTITUTE MATERIALS OR THEIR MIXTURES

**Autografts • Allografts • Bovine Bone • Synthetics**

# botiss grafter

Perforated cap enables optimal  
**hydration within the applicator**

**Efficient application** of the desired amounts  
of bone substitute materials

**Application of up to 2 ml**  
of particulate bone substitute materials

**Comfortable and clean handling**

COMPACT **INTUITIVE DESIGN**



**Provided sterile**


Designed for **single use only**

**Ideal** for maxgraft<sup>®</sup>, cerabone<sup>®</sup>, maxresorb<sup>®</sup>

Allows **mixing of materials**



# workflow

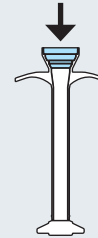


**FILLING**  
via integrated funnel




**INSERTION**  
of plunger rod




**HYDRATION & EJECTION**  
of excess liquid  
(avoid compression of granules)

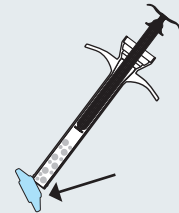

**REMOVAL** of  
perforated **CAP**

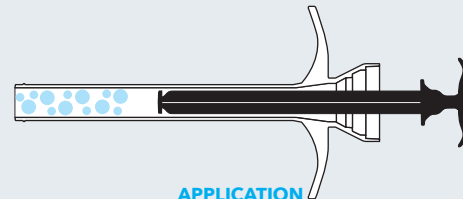

**APPLICATION**  
of bone substitute material

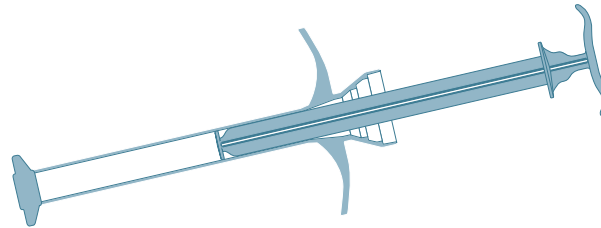

<b>Art.-No.</b>	<b>Content</b>
550001	1 x botiss grafter



botiss biomaterials GmbH  
Hauptstr. 28  
15806 Zossen / Germany

Tel: +49 33769 / 88 41 985  
Fax: +49 33769 / 88 41 986

[www.botiss.com](http://www.botiss.com)  
#botissdental